

GUIDE TO USING ezBG

This guide is a summary only and intended as a quick reference guide only. Please refer to package insert for full instructions and precautions.

To get the most out of using your ezBG, you will need to have set up your Insulin Sensitivity Factor (ISF) and blood glucose targets in the advanced setup menu first.

Select Bolus from the main menu.



Use the down key until the yellow bar is over the word ezBG. Press **OK**.

THIS MUST ONLY BE DONE IF THE BG VALUE IS ABOVE THE TARGET.



The cursor will be flashing on the number beside actual. Use the up key to scroll up to the blood glucose level taken from the meter. Press **OK**.

The cursor will shift to **Show Result**. Press **OK**.



The cursor will be flashing over the number **0.00U**. Using the up and down keys enter in the amount that matches the total on the line immediately above. Press **OK**.

The cursor will be over the word **Go**. Double check that the number dialled up matches the total suggested by the pump. If it does, Press **OK**.

If the number does not match, wait for the pump to go sleep and start again.