

CANNULA FILL GUIDELINES

This information is provided for reference only. Always refer to the instructions for use provided with the infusion set to verify cannula fill requirements.

- ✿ The cannula fill replaces the empty space with insulin to ensure there is no interruption to your basal delivery.
- ✿ You do not need to fill your cannula every time you reconnect to the pump. Cannulae need to be filled whenever a new site is inserted.
- ✿ It is important to remember that different types of sets require different amounts of insulin to fill the cannula.
- ✿ The cannula fill is programmed into the pump through the PRIME/REWIND section of the pumps main menu. Contac Detach do not have cannula fills, they are primed completely prior to insertion.

INSET II



6mm cannula length Fill empty cannula with 0.30 u of insulin

9mm cannula length Fill empty cannula with 0.50 u of insulin

INSET 30



13mm cannula length Fill empty cannula with 0.70 u of insulin

COMFORT



13mm cannula length Fill empty cannula with 0.70 u of insulin

17mm cannula length Fill empty cannula with 0.70 u of insulin

CONTACT DETACH



6mm cannula length Completely prime cannula prior to insertion by priming

8mm cannula length Completely prime cannula prior to insertion by priming