

FILLING A CARTRIDGE FOR THE ANIMAS IR200/2020 PUMPS



Wash hands and open pouch leaving contents on the packet



Insert the blue handle into the extended plunger and turn so the barbs lock in place.



Slide plunger back and forth 3 times to lubricate the inside of cartridge.



Attach filling needle using a clockwise motion.



If using pen insulin cartridges, insert filling needle through rubber septum of the pen cartridge. Holding the pen cartridge use the needle cover to push down on rubber bung at the end of the pen cartridge filling the pump cartridge. Put needle cover back over needle.

If using an insulin vial, pull plunger back to allow maximum air into vial. Insert filling needle through rubber septum of vial and empty all air into the vial. Pull back on the blue handle to fill the pump cartridge. Put needle cover back on needle.



Check for air bubbles. If there are any, ensure that the cartridge is straight up and down and vortex the cartridge or gently tap on the side of the cartridge to move the bubble toward the top.

Push on the blue handle to force the bubble out ensuring the bubble is completely pushed all the way out.



Remove filling needle, using an anti-clockwise motion



Replace with protective cap



Pinch paddles on blue plunger to release

